
## **OXFORD / HOYER ARISE PARTS MANUAL**

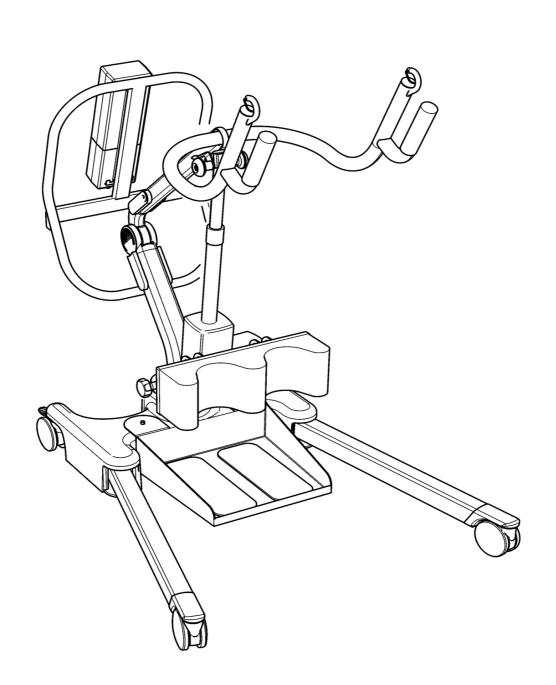



## Sunrise Medical LTC Limited

High Street, Wollaston, West Midlands, DY8 4PS


Tel: +44 (0)1384 44 66 22 Fax: +44 (0)1384 44 66 01 sunmail@sunmed.co.uk www.sunrisemedical.co.uk

## Sunrise Medical LTC Inc

5001 Joerns Drive, Stevens Point Wisconsin, 54481-5040 Tel: (715) 341 3600 (800) 826 0270 (Toll Free)

Rev. C

Oxford/Hoyer Arise 294000-10011

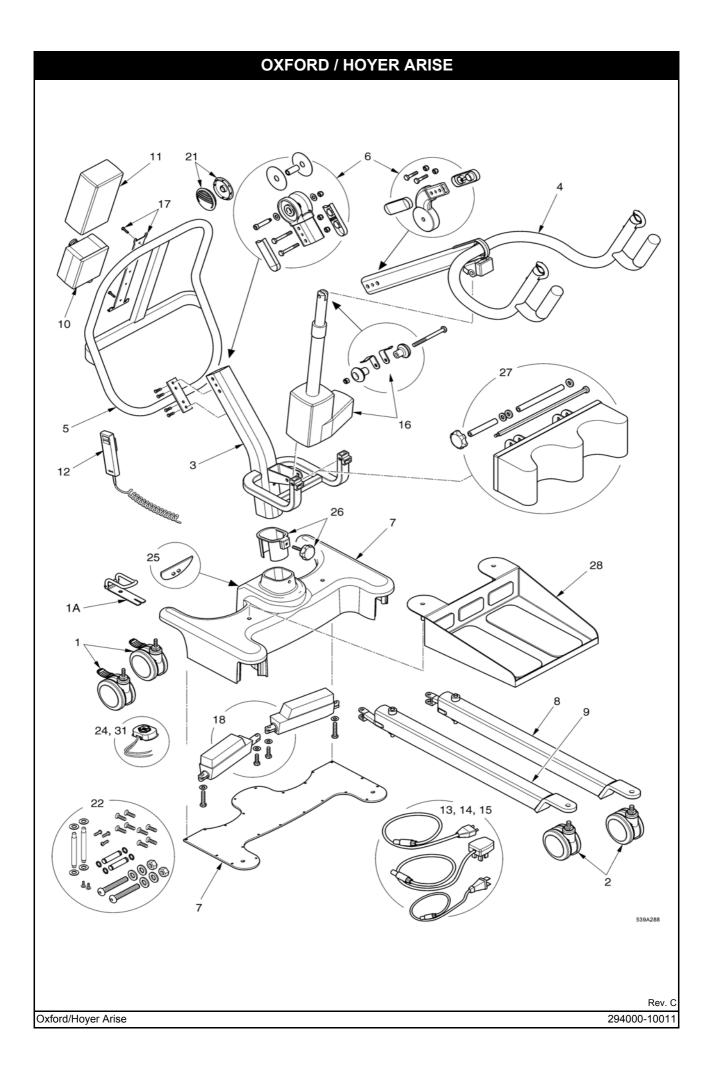

| OXFORD / HOYER ARISE |             |   |     |
|----------------------|-------------|---|-----|
| N°                   | PART NUMBER | DESCRIPTION   | QTY |
| 1                    | 0Y0066      | 100mm rear castor (braked)  | 2   |
| 1A                   | 0Y0263      | Straight-line steering device   | 1   |
| 2                    | 0Y0073      | 100mm front castor (non braked) including tightening wrench (wrench not shown)        | 2   |
| 3                    | 0Y0094      | Mast extrusion including labels   | 1   |
| 4                    | 0Y0095      | Boom extrusion - Cow horn style (including boom end cap)                              | 1   |
| 5                    | 0Y0096      | Push handle assembly (including bolts & fixings)                                      | 1   |
| 6                    | 0Y0103      | Casted boom to mast pivot joint (including connection bolts/fixings/spacers)          | 1   |
| 7                    | 0Y0092      | Base assembly (including labels, base casting and bottom plate, not internal parts)   | 1   |
| 8                    | 0Y0099      | Left leg extrusion (including fixings)  | 1   |
| 9                    | 0Y0100      | Right leg extrusion (including fixings)   | 1   |
| 10                   | 0Y0353      | Linak controller (4 way)  | 1   |
| 11                   | 0Y0067      | Linak battery   | 1   |
| 12                   | 0Y0075      | Linak hand control (4 way)  | 1   |
| 13                   | 0Y0071      | Linak charge lead (UK)  | 1   |
| 14                   | 0Y0114      | Linak charge lead (US)  | 1   |
| 15                   | 0Y0072      | Linak charge lead (EU)  | 1   |
| 16                   | 0Y0097      | Linak actuator - 310mm stroke (inc. location pins, sleeves, washers, nuts and labels) | 1   |
| 17                   | 0Y0069      | Linak battery and controller brackets/fixings   | 1   |
| 18                   | 0Y0093      | Linak leg opening actuator - 35mm stroke (including bolts and fixings)                | 1   |
| 19                   | 0Y0104      | Decal set (Oxford) not shown - no serial number labels                                | 1   |
| 20                   | 0Y0098      | Decal set (Hoyer) not shown - no serial number labels                                 | 1   |
| 21                   | 0Y0126      | Sunrise Pivot Caps  | 2   |
| 22                   | 0Y0102      | Fixings kit (all bolts, screws and fixings)   | 1   |
| 23                   | 0Y0070      | Handset clip (not shown)  | 2   |
| 24                   | 0Y0077      | Mast to base electrical connection kit (including wires)                              | 1   |
| 25                   | 0Y0078      | Foot Push Pad assembly complete   | 2   |
| 26                   | 0Y0112      | Mast locking knob assembly (including plastic base insert)                            | 1   |
| 27                   | 0Y0105      | Knee pad assembly (including all bolts and fixings)                                   | 1   |
| 28                   | 0Y0101      | Foot plate assembly   | 1   |
| 29                   | 0Y0128      | Castor tightening wrench (not shown)  | 1   |
| 30                   | 0Y0053      | Linak charger and stand assembly (not shown)  | 1   |
| 31                   | 0Y0129      | Base to actuator electrical connection kit (including wires)                          | 1   |
| 32                   | 0Y0108      | Boom to mast pivot plastic caps   | 2   |
| 33                   | 0Y0076      | Mast side caps  | 2   |