


LOCO-311 LOCOMOTOR Rotastand...SOLO

ENSURE THESE INSTRUCTIONS ARE READ AND FULLY UNDERSTOOD BEFORE USE.
PLEASE KEEP IN A SAFE PLACE FOR FUTURE REFERENCE.

HEALTH AND SOCIAL SERVICES -
THESE INSTRUCTIONS SHOULD BE LEFT WITH THE USER OF THE LOCOMOTOR ROTA STAND
TO CONSULT AS REQUIRED.

The Rotastand...SOLO is packed in a strong purpose built carton, to ensure safe delivery. Please retain the carton if possible. Inside this carton you will find the Rotastand...SOLO and instructions.

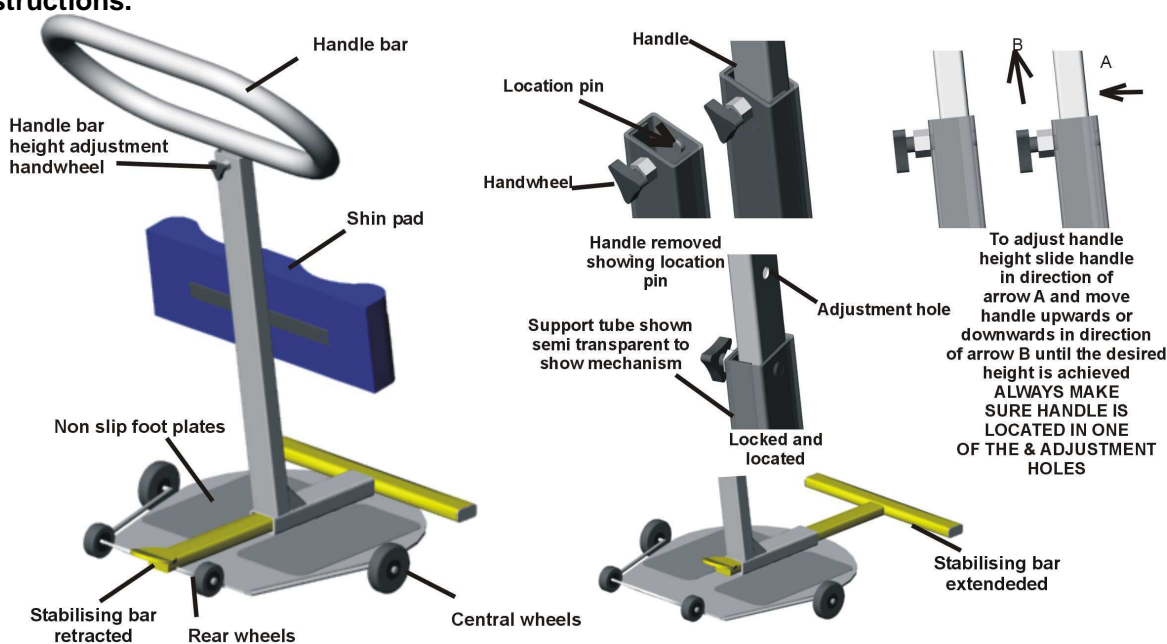

1. Getting Started with your Locomotor Rotastand...SOLO

- 1.1. Remove all items from the carton.
- 1.2. Identify the items against the packing list.

2. Assembly of your Locomotor Rotastand...SOLO (See diagram)

- 2.1. The Rotastand...SOLO comes fully assembled and ready to use

3. Using the Locomotor ROTA STAND .

- 3.1. The Rotastand...SOLO has been designed to allow staff to transfer a patient/client from one seated position to another. Before a transfer can take place staff should satisfy themselves through an appropriate risk assessment that the patient/client is able to bear weight independently
- 3.2. Before using your Rotastand...SOLO adjust the height of the handlebar and the shin pad to suit the patient/client. The shin pad can be moved up or down by 40mm by removing the two fixing screws and attaching it using the two alternative fixing holes. The Rotastand...SOLO can be manoeuvred on its rear wheels and brought to position in front of the patient/client. Assist the patient/client to the front of their chair, ready to stand and place the patient/client's feet on the non slip foot plate and gently wheel the Rotastand...SOLO up to the patient/client on its central wheels. The Rotastand...SOLO should end up with the patient/client's shins firmly against the shin pad.



Select Healthcare (UK) Ltd
Unit 2 Albert Road
Rushden
Northants

NN10 0BU Tel: 01933 411729 Fax: 01933 410213

Designed and Developed by: Proto Magic Innovations Ltd Page 1 ISSUE 8 Issue 9 ROTA solo instructions.doc 16/09/2010

LOCO-311 LOCOMOTOR Rotastand...SOLO

From this position ensure stabilising bar is fully extended and encourage the patient/client, using the handlebar, to pull themselves into a standing position. The nurse/carer should position themselves at the side of the patient/client with one hand on the handlebar ready to assist if required. Once in a standing position, ensure stabilising bar is fully retracted and gently rotate the **Rotastand...SOLO** to the position required ensure stabilising bar is fully extended before encouraging the patient/client to sit, reversing the above procedure.

- 3.3. The **Rotastand...SOLO** can also be used in the same way as a conventional **Rotastand**. In this way the carer would counterbalance the **Rotastand...SOLO** from the front by holding the handle bar, and placing a foot on the base plate as the patient/client brings themselves to a standing position. The stabilising bar is thus not needed to perform this transfer

NB ALL PATIENT HANDLING TASKS SHOULD BE CARRIED OUT AFTER A FULL RISK ASSESSMENT OF THE PATIENT AND THE ENVIRONMENT IN WHICH THE TASK IS TO BE CARRIED OUT.

4. Maintaining and Cleaning your Locomotor Rotastand...SOLO.

- 4.1. Only use a non-abrasive cleaner or disinfectant on the surfaces on the **Rotastand...SOLO**. i.e. Isopropyl alcohol at 70% concentration.
- 4.2. Before every use always ensure that the handlebar and knee pad height adjustment means are fully tightened and that the brake and stabilising bar are functional. Always be vigilant and check for damage
- 4.3. Annual servicing is recommended. See service instructions.
- 4.4. Warranted .for 12 months

5. Some Do's and Don'ts.

- 5.1. **DO** ensure that you have read these instructions and understood them fully, including the glossary of symbols in the bottom of these instructions
- 5.2. **DO** ensure that you carry out an inspection of the Locomotor **Rotastand...SOLO** on a regular basis.
- 5.3. **DON'T** let children play with the Locomotor **Rotastand...SOLO**.
- 5.4. **DON'T** exceed the safe working load of a 200kgs person (31 stones).
- 5.5. **Always** ensure stabilising bar is fully extended before standing or sitting a client

TECHNICAL SPECIFICATION

SAFE WORKING LOAD	200Kg
OVERALL LENGTH	Min 560mm Max 730mm
WIDTH	520mm
HEIGHT	Min 820mm Max 1310mm
WEIGHT	14.5Kg
CONSTRUCTION	Nylon coated stainless steel & mild steel

GLOSSARY

Attention consult accompanying documents before use.



Select Healthcare (UK) Ltd
Unit 2 Albert Road
Rushden
Northants
NN10 0BU Tel: 01933 411729 Fax: 01933 410213

Designed and Developed by: Proto Magic Innovations Ltd Page 2 ISSUE 8 Issue 9 ROTA solo instructions.doc 16/09/2010