

### Comfort Settings on the Pump

As people who use these products are all different body shapes each one will distribute their weight differently i.e. amputee patients. The setting's on the pump are comfort settings but can be broken down to a rough weight guide please remember to check the patients pressure point regularly.

To ensure the patient is receiving adequate care, using your hand check the patient is not bottoming out under high areas of load. If the patient is sat up on the mattress higher pressure may be needed to support them.

### Working Weight Range

**Squirrel Apollo 9000** Max weight 17 stones

Weights in Stones

	S E T T I N G S							
	Weight Settings							
<b>Setting</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>
Stones	4	6.5	8.5	10.5	12	13.5	15.5	17
Killo's	25	41	53	66	76	85	98	107

**Squirrel Luna 9100**

Max weight 25 Stones

	S E T T I N G S							
<b>Setting</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>
<b>Stones</b>	5	8.5	11.5	15	18	21	23	25
<b>Kilo's</b>	31	53	73	95	114	133	144	160

### Squirrel Juno 9200

Max Weight 25 Stones

#### SETTINGS

Setting	1	2	3	4	5	6	7	8
Stones	5	8.5	11.5	15	18	21	23	25
Kilo's	31	53	73	95	114	133	144	160

### Squirrel Topaz 9240

pressure range 30 – 60 MM/ Hg

Max weight 160Kg

#### SETTINGS

Setting	30	35	40	45	50	55	60
Stones	4	8	12	16	20	23	25
Kilo's	25	50	76	101	127	146	160

### Squirrel Diamond/Sapphire/ Emerald 9250

Pressure range 30-60 MM/Hg

Max weight 170Kg

**SETTINGS**

<b>Setting</b>	30	35	40	45	50	55	60
<b>Stones</b>	4	8	12	16	20	24	27
<b>Kilo's</b>	25	50	76	101	127	152	171